DE/EN

**GEBRAUCHS
ANWEISUNG
INSTRUCTIONS
FOR USE**

Hy-Pneumax

CSS
MICRO SYSTEMS

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WEEE DE94635635

Table of Contents

1. Introduction	29
2. Intended use	29
4. Safety instructions	30
5. Cleaning and disinfection	32
6. Scope of delivery	33
7. Device Buttons, Ports, Display	34
8. Use of the device	36
Preparations on the tripod:	37
Mounting accessories on the tripod / articulated arm:	38
Mounting Hy-Pneumax on tripod / mounting arm:	40
9. Settings	42
Normally open or normally closed contact (optionally adjustable):	44
Mono or stereo output wiring:	44
10. Maintenance / Review	46
Hygiene instruction	46
Possible error causes	49
11. Support	50

12. Technical specifications	50
13. Conformity / Standards	51
14. Environmental protection & disposal	51

1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

2. Intended use

The **Hy-Pneumax** pneumatic sensor system is intended for people who, due to their limitations, can only use their mouth to sip and/or puff and not operate other switches.

Hy-Pneumax is used to control call systems (especially **AssistX** call systems) and environment control units and communication devices.

Safety instructions!

Important instructions!

4. Safety instructions

Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification. The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personnel.

Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

EN

Repair

Under no circumstances should you open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

Interaction with medical devices

The device and its components do not contain any active electronic components. The device should nevertheless for security reasons not be placed on a person or brought closer than 15 cm to people using other active medical devices.

The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers.

If necessary, please contact the device manufacturer of the other active medical device.

5. Cleaning and disinfection

Before cleaning the device, disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

 Note the **hygiene instructions** on p. 46ff!

6. Scope of delivery

- **Hy-Pneumax**
- jack cable 1x 3,5mm mono
- bracket for tripod or mounting arm
- screwdriver Philips no. 1
- silicone hose, bacteria/virus filter, mouthpiece
- Instruction for Use
- tripod with weight / mounting arm

EN

7. Device Buttons, Ports, Display

1 Hy-Pneumax sensor

2 bacteria/virus filter

3 silicone hose

4 flex arm

5 mouthpiece

6 weight

8. Use of the device

Connect the **Hy-Pneumax** switching module to the air hose. Connect the mono jack cable to the socket (the stereo jack cable is only intended for special settings!). Plug the other end of the jack cable into the device to be controlled.

Hy-Pneumax triggers a contact when puffing into the mouthpiece. For other settings (e.g. triggering when sipping and puffing), see 9. settings.

 The Hy-Pneumax switching module must be attached to the stand or wheelchair in such a way that the air socket with the bacteria and virus filter points downwards after installation. It must also be ensured that the mouthpiece and the flexarm lead from top to bottom to the operator's lips.

Preparations on the tripod:

For mounting on the tripod, the tripod has to be set up in a stable manner first. To do this, fix the weight with the enclosed locking pin. To do this, pull the stand tube all the way up until you can see the hole for the cotter pin. Then insert the cotter pin to secure the weight. In addition, the boom is screwed onto the stand tube.

EN

Mounting accessories on the tripod / articulated arm:

To use the **Hy-Pneumax**, the mouthpiece, hygiene filter and air hose must be checked, cleaned and replaced regularly (see page 46ff).

- ***Mount the air hose***

Align the flexible hose on the boom of the tripod straight and pull the air hose through the complete boom.

- ***Mount the mouthpiece***

Place the mouthpiece on the air hose and push both together into the flexible hose until everything is firmly in place.

- ***Mount the hygiene filter***

Attach the short adapter hose piece to the air socket of the

Hy-Pneumax switching module. The air hose leading out of the boom is then attached to the bacteria/virus filter. Finally, plug the end of the hose from the hygiene filter onto the air hose adapter.

EN

Mounting Hy-Pneumax on tripod / mounting arm:

The following applies to assembly and disassembly:

- ① ***Mount the Hy-Pneumax on the bracket***
First the bracket is attached to the tripod or the mounting arm. To do this, press the holder with the pipe clamps first onto the tripod / mounting arm tube until the holder snaps onto the tube. Now put the Hy-Pneumax switching module on the bracket.
 - ② ***Remove the Hy-Pneumax from the bracket***
When dismantling, first loosen the Hy-Pneumax switching module from the bracket and then pull the bracket off the tube.
-

1

EN

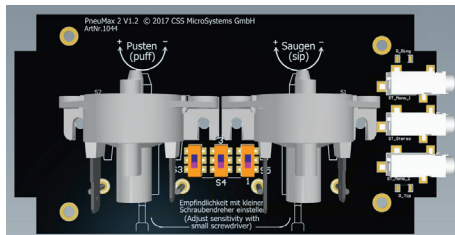
2

9. Settings (Technical staff only!)

Hy-Pneumax can be sensitively adjusted to the individual requirements using adjusting screws. The sensitivity for sipping and puffing can be set separately.

To open the cover, unlock the individual snap strips on each narrow side of the housing by pressing inwards on the left and right with a small screwdriver, turning a little bit while lifting the cover. The set screws inside can be operated with the supplied screwdriver:

 Important!

- First, gently move the screwdriver back and forth to see if it engages in the Phillips head of the screw. The set screws have a certain amount of friction, so you have to apply light pressure to turn them.
- Turn the screwdriver slowly and with moderate force. You can feel when the adjusting screw can follow the movement properly.

EN

 In any case, avoid damaging the cross recess in the plastic screw! If the cross slot is deformed, the pneumatic sensor usually has to be replaced!

- Turning in the adjusting screw (clockwise **[+]**) makes the pressure switch more sensitive for both sipping and puffing: You have to sip / puff less strongly to trigger. Unscrewing the adjusting screw (counterclockwise **[-]**) makes the switch less sensitive.

Depending on your personal disposition, the optimal switching point can be set.

Normally open or normally closed contact (optionally adjustable):

Hy-Pneumax can be used with both closing and opening contacts:

- ***Normally open contact (NO)***

The contacts are in normally open mode at the factory, the settings for sipping and puffing are as described above.

- ***Normally closed contact (NC)***

The switching behavior of the Hy-Pneumax can also be reversed if necessary. To do this, simply turn the adjusting screws in so far that the switches have permanent contact. The closing puffing contact is now an opening sipping contact and the previously closing sipping contact now becomes an opening puffing contact.

Mono or stereo output wiring:

A mono jack cable is included (Stereo cable (3pin) is not included). Hy-Pneumax has 2 functions (sip and puff), which are switched to the output socket(s). The middle socket is fitted as standard. This can

either be wired with both functions as a mono output or with both functions separately. To do this, the dip switch S4 is set to:

Function	S3	S4	S5
Middle socket: (default) Reacts to puffing, with stereo cable to sipping and puffing (to center and tip contact)			
Middle socket with mono normally open function NO: Sipping and puffing brought together on one contact. Air switches must be set to normally open mode.			
Middle socket with mono break function NC: Sipping and puffing brought together on one contact. Air switches must be set to normally closed mode.			
Outer sockets (if equipped): Both sockets with a common contact.			
Both sockets without common contact.			

10. Maintenance / Review

Hy-Pneumax contains components that have to be replaced or cleaned regularly to ensure perfect hygiene. You must follow the hygiene instructions below. In addition, the following checks must be carried out every three months:

- function
- plug connector for damage and contamination
- damage to the casing and other components

Hygiene instruction

The sensor, which can be operated with sipping and puffing, consists of three interchangeable components that have to be replaced or cleaned regularly to ensure perfect hygiene.

- ***Bacteria/virus filters:***
The bacteria/virus filter must be exchanged for a new filter
-

every time the user changes.

The filter must also be replaced at least once a month for the same user, or more often as instructed by the doctor. It is not possible to clean the bacteria/virus filter.

- ***hose:***

The hose must be exchanged for a new hose **every time the user changes**.

The hose must also be replaced at least **every 14 days for the same user**.

If the user suffers from a **respiratory tract infection**, the hose must be replaced at least **every three days**, or more often as instructed by the doctor. It is not possible to clean the hose.

- ***Mouthpiece:***

The mouthpiece must be exchanged for a new mouthpiece **every time the user changes**.

The mouthpiece must also be exchanged **every 14 days for the same user**, or more often as instructed by the doctor.

The mouthpiece can be cleaned with the usual disinfection methods, it is heat-resistant up to 250 °C.

If the user suffers from a **respiratory infection**, the mouthpiece must be replaced at least **every three days**, or more often as instructed by the doctor. Disinfection is possible as described above.

 Use only original parts from CSS MicroSystems!

Possible error causes

Symptom	Cause
No function/ a function does not work	The bacteria and virus filter could be full of saliva. Check whether the air hose is possibly filled with liquid. If so: Has the Hy-Pneumax been correctly installed with the air nozzle pointing downwards? If this has been installed the wrong way round and there is liquid in the hose, the bacterial and viral filter must be replaced. The mouthpiece and the gooseneck / flexible tube must lead from top to bottom to the operator's lips, otherwise the bacterial and virus filter will fill up.
Hy-Pneumax has an uncontrollable permanent function	Most likely, one of the pneumatic switches is set incorrectly: If the adjusting screw of a pneumatic switch is turned clockwise, the switch is always easier to trigger. If the adjusting screw is screwed in too far, the switch gives permanent contact. Remedy: Unscrew the adjusting screw until the permanent contact is interrupted.
Hy-Pneumax has Permanent function and is no longer adjustable	The adjusting screw on the pneumatic switch is probably broken. This can happen if the screw (plastic) is turned with the wrong tool or with too much force. If the cross recess of this plastic screw is destroyed by force, it can no longer be adjusted. The only thing that helps here is an exchange.

11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to support@csslabs.de. Please be sure to include a description of the problem and the serial number of the device.

12. Technical specifications

Dimensions: 123 x 68 x 45 mm

Weight: 150 g

IP Rating: IP 54

Power supply: no active power supply needed

Connection: 3.5 mm stereo jack socket, silicone hose

Ø inside 3 mm, outside 6 mm

Temperature ranges

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

13. Conformity / Standards

CE This product is CE compliant.

EN

Hy-Pneumax basically meets the requirements of DIN VDE 0834 in combination with **AssistX** call systems.

14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

C.S.S
MICRO SYSTEMS

ASSISTIVE TOOLS FOR A BETTER LIFE

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© 2026 CSS MicroSystems GmbH