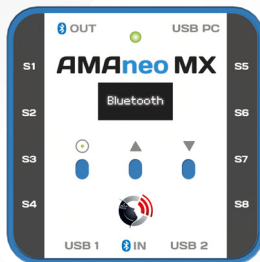
DE/EN

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**INSTRUCTIONS
FOR USE**

AMAneo MX

*****Preliminary *****

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WEEE DE94635635

AMAneo ® is a registered trademark of CSS MicroSystems GmbH.
Bluetooth ® is a registered trademark of BLUETOOTH SIG, INC.

Contents

1. Introduction	36
2. Intended Use	37
3. Explanation of Symbols	37
4. Safety Instructions	38
5. Cleaning and Disinfection	40
6. Scope of Delivery	41
7. Controls, Connections, Indicators	42
Status LED table:	43
8. Commissioning and Operation	44
Charging the battery:	44
Switching the device on and off:	44
Selecting the language:	45
Selecting a profile set:	45
Connecting AMAneo MX to a PC, notebook, tablet, or smartphone: ...	45
Connecting AMAneo MX to input devices:	46
Standard profile sets:	46
Overview of predefined profile sets:	47

9. Settings	58
Switching the device on and off:	58
Menu:	58
Profiles:	59
Bluetooth:	59
Advanced:	60
10. Setting up Profile Sets	61
11. Maintenance / Inspection	63
12. Support	63
13. Technical Data	64
14. Conformity	64
15. Environmental Protection and Disposal	65

1. Introduction

Thank you for choosing a product from CSS. Please read these Instructions for Use carefully and follow the relevant instructions when operating the device. Please keep these Instructions for Use readily accessible at the location where the device is used.

2. Intended Use

AMAneo MX is a HID multi-adapter that allows functions to be assigned to the outputs based on the devices and control elements connected to the inputs. The device provides eight 3.5 mm mono jack sockets for common switches or sensors, two USB-A ports, and a **Bluetooth®** interface for HID devices such as mice and keyboards as inputs. For connection to computers, tablets, or smartphones, a USB-C port and an additional **Bluetooth®** interface operating as a HID device are available as outputs.

Using profiles, it can be defined how each input device behaves and which function it performs at the output. Standard profiles are pre-configured on the **AMAneo MX** at delivery. Custom profiles can be created or modified using the configuration software, which is available as a free download.

3. Explanation of Symbols

Safety information!

Important information!

4. Safety Instructions

Handling

Please handle the device with care. The device contains sensitive electronic components. No modifications may be made to the device.

The device is not life-sustaining or life-supporting. The device consists of several components. Small parts may become detached, which could pose a choking hazard or other danger to children. Small children or persons with cognitive impairments should not have access to or use the device without supervision. Do not use the device in damp, wet, extremely cold, or extremely hot environments (*see temperature specifications in the Technical Data section*).

Repair

Under no circumstances should you attempt to open or repair the device yourself.

Disassembling the device may cause permanent damage to the device and may result in injury. Contact your specialist dealer or the manufacturer if the device is damaged or does not function properly. If the device or its accessories are mechanically damaged, it must no longer be used.

Interaction with medical devices

The device and its components may emit electromagnetic radiation. For this reason, the device must not be placed on a person or brought closer than 15 cm to persons who use active medical devices. The device must also not be placed directly on active medical devices.

Please pay particular attention to this information in the case of persons with implanted pacemakers. If necessary, consult the manufacturer of the active medical device.

5. Cleaning and Disinfection

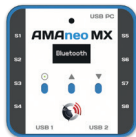
Before cleaning the device, switch it off and disconnect all cables. If necessary, clean the device using a mild cleaning agent. Cleaning should only be carried out using a wiping cloth. Wipe all external surfaces and allow them to air dry.

Do not use sprays that may allow moisture to enter the device. Under no circumstances must any component of the device be subjected to steam or wet sterilisation.

6. Scope of Delivery

- A) **AMAneo MX**
- B) USB-C to USB-A cable
- C) Instructions for Use

A)


B)

C)

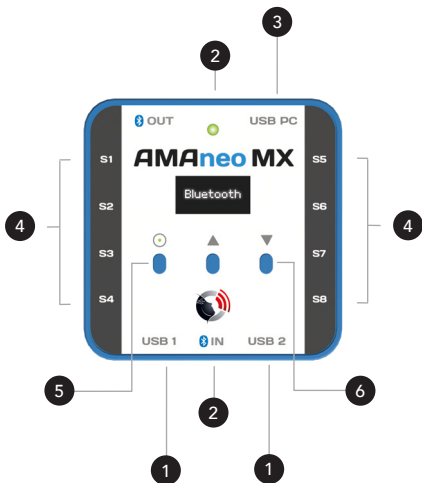7. Controls, Connections, Indicators

- 1** USB ports for HID devices, e.g. mouse or keyboard

- 2 Bluetooth® (BLE) interfaces, 1 × input, 1 × output
- 3 USB port for PC, notebook, tablet, charger
- 4 8 × 3.5 mm mono jack sockets
- 5 On/Off, OK for menu control
- 6 Adjustment for menu up/down and functions, e.g. +/-

Status LED table

Red	Blue	Green	Meaning
blinking slowly			Battery operation, battery < 20%
	blinking fast		Bluetooth search (scan)
	blinking slowly		Battery operation and Bluetooth host connected
	permanently on		USB charging and Bluetooth host connected
		blinking slowly	Battery operation and Bluetooth host not connected
		permanently on	USB charging and Bluetooth host not connected

8. Commissioning and operation of the device

For initial commissioning, first charge the battery, select the language, select a profile set, and connect the device to be controlled.

a) Charging the battery:

The **AMAneo MX** is equipped with an integrated battery. The device is charged via the USB port ③. The **AMAneo MX** can be charged using the supplied USB-C to USB-A cable via a mains power adapter or directly via a PC. Alternatively, a smartphone charger with an available USB-C cable can be used.

Please charge the battery for approximately 1–2 hours.

b) Switching the device on and off:

To switch on the device, briefly press the button ⑤. A beep will sound and the device will switch on. To switch off the device, press and hold the button ⑤ for at least 5 seconds until the progress bar on the display has completed. After releasing the button ⑤, the device is switched off.

c) Selecting the language:

Various languages are displayed on the screen. Please select the appropriate language using the arrow buttons **6** and confirm with the button **5**.

d) Selecting a profile set:

During initial commissioning, a profile set suitable for the intended use must first be selected (see *“Standard profile sets” on the following pages*). Depending on the intended use of the AMAneo MX, select the appropriate profile set using the arrow buttons **6** and confirm with the button **5**.

The profile sets can be changed at any time (see section 9. Settings).

e) Connecting AMAneo MX to a PC/notebook/tablet/smartphone:

The **AMAneo MX** can be connected to a PC, notebook, tablet, or smartphone using the supplied USB cable or an available USB-C cable. To do so, connect the USB-C plug of the cable to the USB port **3**. Alternatively, the **AMAneo MX** can be connected to the devices (*hosts*) via Bluetooth®. To do this, pair **AMAneo MX** in the Bluetooth settings of the respective device. No additional settings are required on the **AMAneo MX**.

f) Connecting **AMAneo MX** to input devices:

The two lower USB ports ❶ are intended for connecting a mouse and/or keyboard. Virtually all standard mice as well as assistive mice, such as head mice, trackballs, etc., can be connected. Connect the required switches to the eight jack sockets. ❷

Bluetooth® mice and Bluetooth® keyboards can also be connected. To do so, press and hold the button ❸ for 2 seconds to open the menu, use the arrow buttons ❹ to select "Bluetooth" and confirm with the button ❸, then select either "Keyboard" or "Mouse" and follow the instructions shown on the display.

Standard profile sets

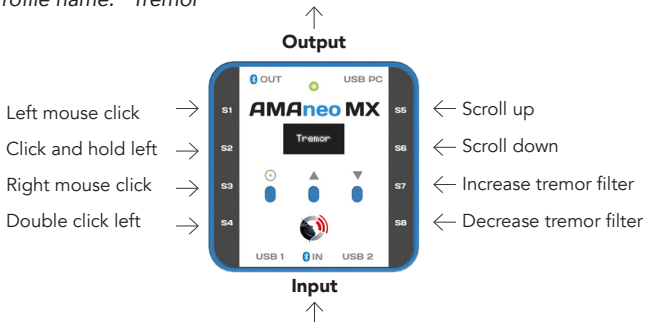
The functionality of the **AMAneo MX** is implemented via 10 profile sets. This means that the basic solution provided by the **AMAneo MX** is determined by selecting a profile set.

Eight predefined profile sets are available, which already cover almost all requirements. In addition, two further profile sets can be individually configured (*see section 10. Setting up profile sets*).

Overview of predefined profile sets:

Profile Set 1: AMAneo MX Anti-Tremor

Profile name: "Tremor"


INPUT	OUTPUT
USB 1/USB 2: Mouse/keyboard	USB PC: Computer, Smartphone, Tablet
Bluetooth: Mouse/keyboard	Bluetooth: Computer, Smartphone, Tablet

Note: Buttons **6** for adjusting the tremor filter (increase/decrease)


Profile Set-2: **AMAneo MX** Switch Mouse

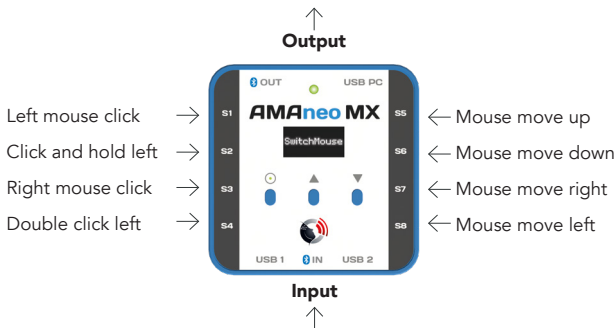Profile name: "SwitchMouse"

INPUT	OUTPUT
Off	USB PC: Computer, Smartphone, Tablet
Off	Bluetooth: Computer, Smartphone, Tablet

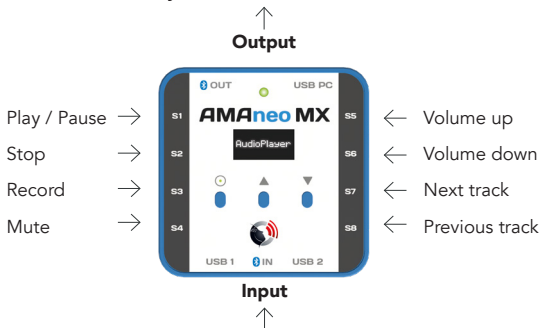
Note: Buttons **6** for adjusting mouse movement speed (faster/slower)

Profile Set-3: AMAneo MX Audio Player*Profile name: "AudioPlayer"*


INPUT	OUTPUT
Off	USB PC: Computer, Smartphone, Tablet
Off	Bluetooth: Computer, Smartphone, Tablet

Profile Set-4: **AMAneo MX** SwitchBox

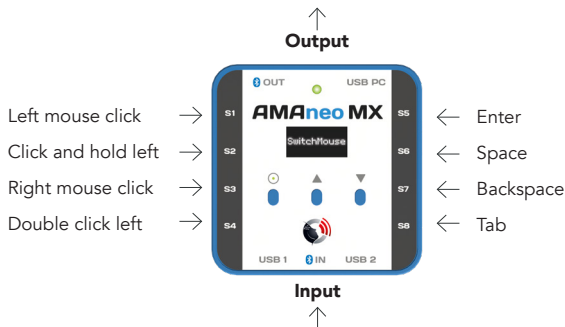Profile name: "Switchbox"

INPUT	OUTPUT
Off	USB PC: Computer, Smartphone, Tablet
Off	Bluetooth: Computer, Smartphone, Tablet

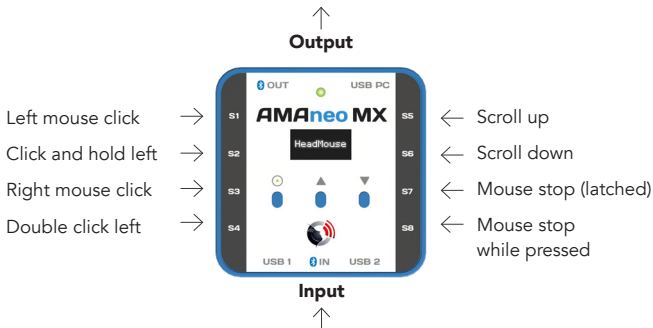Profile Set-5: AMAneo MX HeadMouse*Profile name: "HeadMouse"*

INPUT	OUTPUT
USB 1/USB 2: Mouse/keyboard	USB PC: Computer, Smartphone, Tablet
Bluetooth: Mouse/keyboard	Bluetooth: Computer, Smartphone, Tablet

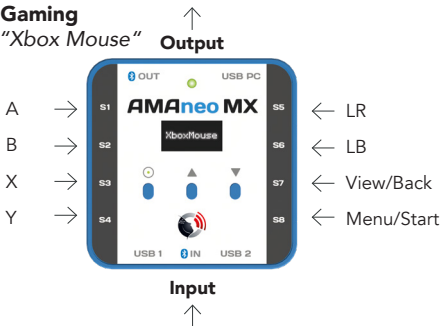
Motion control: Key M on a connected keyboard

Profile Set-6: **AMAneo MX** Xbox Adaptive Controller Gaming

Warning: A Microsoft Adaptive Controller is required between the **AMAneo MX** and the Xbox.

Xbox Mouse Gaming

Profile name: "Xbox Mouse" **Output**


INPUT	OUTPUT
USB 1/USB 2: Mouse	Game Pad USB
Bluetooth: Mouse/keyboard	Bluetooth: Computer, Smartphone, Tablet

Note: Buttons **6** for switching profiles within the profile set (press for 2 seconds) Motion control: Key M on a connected keyboard

Xbox Button Gaming

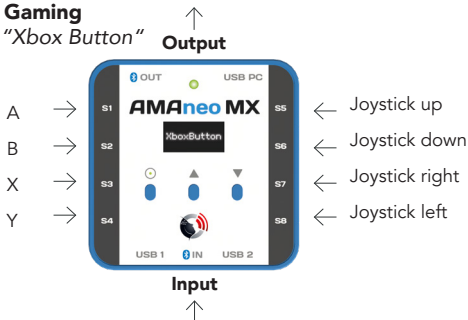Profile name: "Xbox Button"


INPUT	OUTPUT
USB 1/USB 2: Mouse	Game Pad USB
Bluetooth: Mouse/keyboard	Bluetooth: Computer, Smartphone, Tablet

Note: Buttons **6** for switching profiles within the profile set (press for 2 seconds) Motion control: Key M on a connected keyboard

Profil Set-6: **AMAneo MX** Windows Gamepad Games Windows Gamepad Joystick

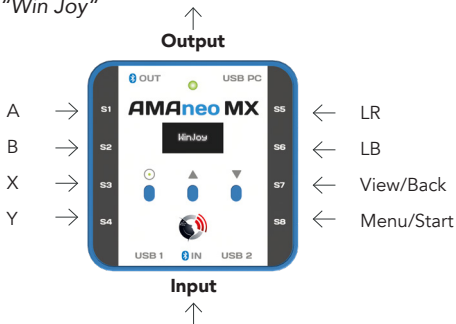Profile name: "Win Joy"


INPUT	OUTPUT
USB 1/USB 2: Mouse	Game Pad USB
Bluetooth: Mouse/keyboard	Bluetooth: Computer, Smartphone, Tablet

Note: Buttons **6** for switching profiles within the profile set (press for 2 seconds) Motion control: Key M on a connected keyboard

Windows Gamepad Buttons

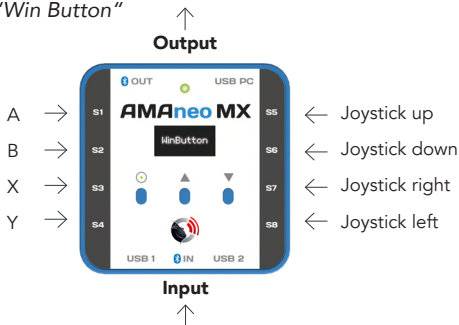Profile name: "Win Button"


INPUT	OUTPUT
USB 1/USB 2: Mouse	Game Pad USB
Bluetooth: Mouse/keyboard	Bluetooth: Computer, Smartphone, Tablet

Note: Buttons **6** for switching profiles within the profile set (press for 2 seconds) Motion control: Key M on a connected keyboard

Profil Set-6: **AMAneo MX** Cursor Key Games Gaming WASD Keys

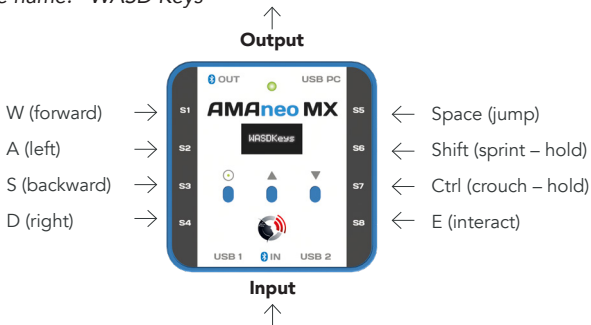Profile name: "WASD Keys"


INPUT	OUTPUT
USB 1/USB 2: Mouse	USB PC: Computer/Laptop
Bluetooth: Mouse	Bluetooth: Computer/Laptop

Note: Buttons **6** for switching profiles within the profile set (press for 2 seconds) Motion control: Key M on a connected keyboard

Gaming Arrow Keys

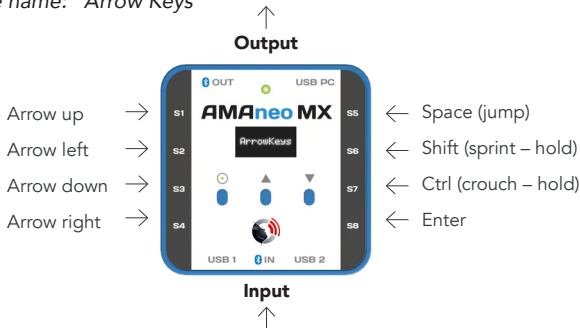Profile name: "Arrow Keys"

INPUT	OUTPUT
USB 1/USB 2: Mouse	USB PC: Computer/Laptop
Bluetooth: Mouse	Bluetooth: Computer/Laptop

Note: Buttons **6** for switching profiles within the profile set (press for 2 seconds) Motion control: Key M on a connected keyboard

9. Settings

Switching the device on and off:

To switch on the device, briefly press the button **5**, a beep will sound and the device will switch on.

To switch off the device, press **5** and hold the button for at least 5 seconds until the progress bar on the display has completed. After releasing the button **5**, the device is switched off.

Settings can be adjusted directly on the device via the display. Additional settings can be configured using the configuration software via profiles (see section 10. *Setting up profile sets*).

Menu:

The **AMAneo MX** has three buttons **5** **6** located below the display. Press and hold the button **5** for 2 seconds to access the menu, where profile changes can be made, a PC / smartphone / tablet can be connected via Bluetooth®, or Bluetooth mice / keyboards can be connected to the **AMAneo MX**. Use the respective button **6** to navigate up and down through the menu items. Once the desired menu item is selected, confirm with the button **5**.

Profiles:

All profile sets currently stored on the device are displayed here. They can be selected and confirmed, after which they become active.

Note: In some profile sets (e.g. *Anti-Tremor*, *SwitchMouse*), specific settings can be adjusted within the profile using the buttons **6**, for example changing the tremor filter strength or adjusting the mouse speed (see also *Standard profile sets*).

Bluetooth:

Bluetooth® mice and **Bluetooth®** keyboards can be connected. In the menu item "Bluetooth", select either "Keyboard" or "Mouse" as appropriate and confirm "Pair". Then confirm "Scan" with the button **5**, a beep will sound. The **AMAneo MX** scans for all **Bluetooth®**-enabled devices in the vicinity and displays them. Use the arrow buttons **6** to select the device to be connected and confirm with the button **5**. A beep will sound. Then select "Connect" using the arrow buttons **6** and confirm with the button **5**.

Note: When connecting a keyboard, a 6-digit code is displayed on the **AMAneo MX**. Please enter this code using the keyboard to establish the connection between the two devices. Once the connection has been established, the **AMAneo MX** will display CONNECTED on the screen.

Advanced:

In this menu item, the language (*menu navigation*) can be changed and a factory reset can be performed.

Warning: Performing a factory reset will delete all individual settings.

Note on connecting game consoles (via appropriate adapters):

To use the **AMAneo MX** with a game console, it is important that GAMEPAD is selected in the profile set. If the **AMAneo MX** is set to Standard (*BT / USB*), it will not be recognized by the game console.

The **AMAneo MX** always requires a Microsoft Adaptive Controller to connect to an Xbox. Settings must be configured on the Xbox. If a Microsoft Adaptive Controller is used with a Nintendo Switch, an additional adapter is required and settings must be configured on the Nintendo Switch. For further information, please refer to the information available at csslabs.de/download.

10. Setting up profile sets

To set up or modify profile sets, the **AMAneo MX** Configuration Software is required. Please download it here: csslabs.de/download Then install and start the software according to the instructions.

Note: When the software is launched again at a later time, it will indicate whether an update is available (*internet connection required*).

Please connect the **AMAneo MX** to the USB port **3** using the supplied USB-C to USB-A cable and connect it to a Windows PC or notebook. The **AMAneo MX** will be automatically recognized by the configuration software.

On the left-hand side, click on the profile to be edited using the mouse. In the "Profile name" field, enter the desired name for the profile. In the adjacent field, select whether the **AMAneo MX** should be recognized as Standard (*BT/USB*) or as a Gamepad by clicking the arrow button. In the "BT" field, select whether Bluetooth should be enabled in order to connect the **AMAneo MX** to the PC. Please click on the respective switch field with the left mouse button.

On the right-hand side, the “Switch settings” field appears together with the corresponding switch number that has been selected. First, select the TYPE. Once the type has been selected, choose the BUTTON. After selecting the button, select the BEHAVIOR. Please repeat this procedure for all eight connections.

At the bottom, select whether a mouse or keyboard is connected via USB or Bluetooth under input devices. Please select and confirm using a left mouse click. When clicking on the “Input functions” button, a separate menu opens in which the tremor filter can be adjusted individually. Once all settings have been configured, click “Save profile” in the upper right corner to save the profile.

For further information, please refer to the information available at csslabs.de/download.

11. Maintenance / inspection

The device normally does not require maintenance. However, we recommend checking the following from time to time:

- Connectors for damage and contamination
- Damage to the housing
- Firmware update (www.csslabs.de/download)

When performing an update, please follow the instructions on our website. The update should only be carried out by qualified personnel.

12. Support

For all technical questions regarding the product, please first contact the specialist dealer from whom you purchased this product. If necessary, if the dealer is not available or cannot provide further assistance, please send us an email at support@csslabs.de. Please be sure to include the error or operating problem and the serial number.

13. Technical data

Dimensions: 74 x 74 x 27 mm

Weight: 95 g

Power supply: USB

Connections: USB-C (USB PC), 2 × USB-A (mouse/keyboard),
8 × 3.5 mm jack sockets (S1–S8), 2 × Bluetooth interfaces

Battery life: approx. 1 day, depending on usage

Temperature ranges:

Operation: 0 °C to 45 °C

Transport and storage: -10 °C to 50 °C

15. Environmental protection and disposal

All electrical and electronic equipment must be disposed of separately from household waste at designated collection points. This product is subject to the European Directive 2002/96/EC and must therefore be disposed of properly at designated collection points for waste electrical and electronic equipment.

WEEE DE94635635

AMAneo ® is a registered trademark of CSS MicroSystems GmbH.
Bluetooth ® is a registered trademark of BLUETOOTH SIG, INC.

RoHS & REACH

The product is compliant and does not contain restricted hazardous substances above the allowed limits.

The product does not contain Substances of Very High Concern (SVHC) above 0.1% weight by weight (w/w) as listed on the current ECHA Candidate List.

The product conforms to the essential requirements of the relevant EU directives. The Declaration of Conformity is available upon request

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions.

(1) This device may not cause harmful interference. (2) This device must accept any interference received, including interference that may cause undesired operation.

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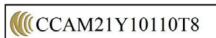
IC (Canada)

This device complies with Industry Canada license-exempt RSS Standard(s). Operation is subject to the following two conditions. (1) This device may not cause harmful interference. (2) This device must accept any interference received, including interference that may cause undesired operation.

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© 2025 CSS MicroSystems GmbH