DE/EN (EU)

**GEBRAUCHS-  
ANWEISUNG**  
**INSTRUCTIONS  
FOR USE**

# **AssistX Mobil Repeater**

**C.S.S.**  
MICRO SYSTEMS

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---

## 1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

## 2. Intended use

The **AssistX Mobil Repeater** is a component for the AssistX Mobil mobile call system. With the repeater, the range of the call system can be increased considerably, especially in larger facilities and buildings. Up to 8 repeaters can be used within one system, each extending approximately by the original range of the transmitter and receiver, which can vary depending on the building structure. Multiple user groups can be used via the same repeater.

## 3. Explanation of symbols

Safety instructions!

Important instructions!

---

## 4. Safety instructions

### Handhabung

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification.

The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personal.

### Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

---

## **Repair**

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

## **Interaction with medical devices**

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using other active medical devices. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

## **5. Cleaning and disinfection**

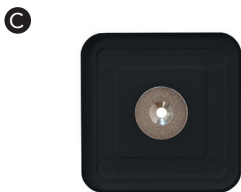
Before cleaning the device, switch it off and disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

---

## 6. Scope of delivery

- A) **AssistX Mobil Repeater**
- B) Power supply
- C) Magnetic wall mount with screw
- D) Instructions of Use

---

## 7. Device Buttons, Ports, Display

---

- 
- 1 Antenna
  - 2 On/Off Button
  - 3 USB port (power supply)
  - 4 Status LED
  - 5 Reset button (behind a hole in the case)
- 

- 4 **Normal operation** lights **green**,  
power supply connected.
- 

- 4 **Emergency operation**
    - ● ● 10%
    - ● ● 50%
    - ● ● voll
-

## 8. Use of the device

**!** Required for operation: The firmware on all components in the system (transmitters, receivers and repeaters) must be the same, at least V1.38!

### Installation:

#### Screw the antenna onto the **AssistX Mobile Repeater** and align it:

Wall mounted:

Table top:

Then plug the supplied power supply into the socket and connect it to the repeater at the **3** USB socket.

Switch on the repeater by pressing the **2** On/Off button for approx. 3 seconds. The **4** LED lights up and you will hear one or more beeps.

The **4** LED shows the charging status (**red, yellow, green**). If the **4** LED lights **green**, the internal buffer battery (emergency power supply) is fully charged. This should always be the case in normal operation. The **4** LED also shows the repeater's radio activity by flashing **blue**.

---

 If the **4** LED flashes **orange** or **red**, please check the power supply and the socket. The repeater is then no longer charged and runs on battery power (emergency operation)! A out-of-range alarm on one of the receivers used can also indicate a power supply problem on the repeater!

---

- 4 Status LED:**
-  Normal operation, power supply connected
  -  flashing: Emergency operation, Battery 100%
  -  flashing: Emergency operation, Battery 50%
  -  flashing: Emergency operation, Battery 10%
  -  flashing: Radio activity of repeater
- 

### Using several repeaters:

If you are using **more than one repeater**, do not switch them on at the same time. Please wait approx. 20 seconds between switching on. The repeaters organize themselves automatically and assign themselves an identifier, which you can recognize by the number of beeps when switching on (1x to 8x). If you try to install more than 8 repeater, you will hear a warning sound from the 9th on.

---

If you need to switch off one or more repeaters during operation (press the ② On/Off button for longer than 3 sec) e.g. to change the location, it is not critical in which order you switch back on the devices, the correct initialization takes place automatically. This is also the case if you need to replace a repeater.

 By pressing the On/Off button for 1 sec, the repeater is restarted and initialized automatically.

If you want to use the wall mount, mount it to the wall using the screws provided. The repeater then only has to be pressed into the wall mount, it is held in place magnetically.

## 9. Settings

No settings are necessary or possible on the repeater. If the device is in an undefined state, you can **very carefully** press the ⑤ Reset Button (behind a hole) with a paper clip or a similarly thin object. The repeater restarts and automatically configures itself as described above.

---

## 10. Maintenance / Review

The device must be checked for safety and functionality every three months. The check must be done by trained specialist personnel only. The following should be checked:

- Battery (replace every 3 years at the latest)
- Connectors for damage and soiling
- Damage to the housing
- Update Firmware

The **AssistX Mobil Configurator** software is available at [www.csslabs.de/download](http://www.csslabs.de/download) (for Windows). You can use it to update the firmware of the repeater via the ③ USB connection (only for specialist personnel!).

 Please note: If you update the repeater, all other components, transmitters, receivers and other repeaters also need to be updated. Otherwise, safety-critical system failures can occur!

---

## 11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to [support@csslabs.de](mailto:support@csslabs.de). Please be sure to include a description of the problem and the serial number of the device.

## 12. Technical specifications

Dimensions: 74 x 74 x 26 mm (without Antenna)

Weight: 130 g

IP Rating: IP 54

Power supply: 5 V , ~100 mA

Frequency: 869,225 MHz (Social Alarm)

Backup Battery: >8 h

### Temperature ranges:

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

## 13. Conformity/Standards

 This product is CE compliant.

### **AssistX Mobil Repeater**

meets the requirements of DIN VDE 0834.

---

## 14. Environmental protection and disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

---

**ASSISTIVE TOOLS FOR A BETTER LIFE**

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© 2024 CSS MicroSystems GmbH