DE/EN (EU)

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**INSTRUCTIONS  
FOR USE**

# BIF8+IE

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---

## 1. Introduction

Thank you for choosing a product from CSS. Please read these Instructions for Use carefully and follow the relevant instructions when operating the device. Please keep the Instructions for Use readily available at the place where the device is used.

## 2. Intended use

The **BIF8+IE** bed interface enables the wireless control of an electrically adjustable care bed by infrared and Easywave radio by means of an environmental control unit. The **BIF8+IE** is plugged into the bed motor control instead of or in parallel with the wired hand control.

For this purpose, the **BIF8+IE** provides up to ten different switching functions.

## 3. Explanation of symbols

Safety instructions!

Important instructions!

---

## 4. Safety instructions

### Handling

Please handle the device with care. The device contains sensitive electronic components. No modifications may be made to it.

The device is not life-sustaining or life-supporting. The device consists of several components. Small individual parts may become detached from the device, which may pose a risk of suffocation or other danger for children. Small children or persons with cognitive impairments should not have access to or use the device without supervision.

Do not use the device in humid, wet, very cold or very hot environments (see temperature specifications in Technical Data).

### Mounting

The control unit (box) must be mounted in such a way that it cannot fall down. Only components from the manufacturer may be used. The user bears the sole risk for the installation. The hand control unit should be hooked onto the side rail or the foot or head section of the nursing bed. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

### Repair

Do not attempt under any circumstances to open or repair the device yourself. Disassembling the device may cause permanent damage to the device and may result in injury. Contact your specialist dealer or the manufacturer if the device is damaged or does not function properly. If the device or its accessories are mechanically damaged, they must no longer be used.

---

## 5. Cleaning and disinfection

Before cleaning the device, disconnect all cables. The device should be cleaned monthly using a disinfectant and cleaning agent intended for use in hospitals. Cleaning should only be carried out with a wiping cloth. Wipe all external surfaces and allow them to dry on their own or follow the wiping instructions. Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

## 6. Scope of delivery

**A) BIF8+IE** bed interface with connected hand control unit (Handset HS plus) and power supply unit

**B) Instructions for Use**

A

B

## 7. Device Buttons, Ports, Display

- 1 Control unit
- 2 Indicator light
- 3 Hand control unit
- 4 Function buttons bed control 1–10
- 5 Button MODE (IR/Easywave)
- 6 Button OPT
- 7 IR receiver/transmitter

### 2 Indicator light

- Green: Ready for operation, in standby mode
- Off: Off / remains off until a new signal is received
- Orange: Receiving valid infrared command
- Red: Error, functions locked
- Flashing red: Receiving command, error: relay contact no longer disconnecting, functions locked

## 8. Use of the Device

### Introduction

The hand control unit Handset HS plus ③ is permanently connected to the **BIF8+IE** control unit via a spiral cable.

It allows control of the bed functions according to the bed functions printed in rows 1 to 5 (④ buttons 1...10) directly by pressing a button (e.g. for nursing staff). The left column is assigned to the "up" functions, the right column to the "down" functions.

At the bottom are the buttons "IR/Easywave" ⑤ (MODE) and "OPT" ⑥. Each button can light up either green or red.

The hand control unit has an **infrared receiver** as well as an **Easywave radio receiver**. Depending on the selected mode, either infrared signals or Easywave radio signals can be forwarded to the **BIF8+IE**.

The hand control unit also has an **infrared transmitter** to transfer IR signals to the controlling device (e.g. environmental control unit or communication device). No external remote control is required for this. Such a remote control is optionally available (CSS Art. No. 20324).

### Commissioning

Mount the control unit ① under the bed in such a way that no cables can be trapped.

**Connect the power supply unit** and then plug the control unit into the bed control using the appropriate connector.

**The hand control unit must be attached directly to the bed** by hooking it onto the side rail or the head or foot board. When using infrared, ensure that the hand control unit is not covered.

---

## Use

By default, the **BIF8+IE** is set for infrared control. It can be used directly with the IR signals stored in many devices for the BIF8+. Otherwise, the corresponding settings must first be carried out as described in section 9 "Settings".

The bed is operated via the user's environmental control unit or communication device.

The hand control unit **3** can always be used independently, e.g. by nursing staff. The bed functions can be operated via the movement buttons 1...10 according to the printed symbols. If an additional (original) hand control unit is available on the nursing bed, it can be used in parallel.

## Safety

The **BIF8+IE** has several safety functions.

### Automatic shutdown of a continuous function

A continuous function activated by pressing and holding a **4** function button is interrupted after a predefined time. The predefined time until automatic shutdown can be set by the technician in one-second steps from 1 to 25 seconds.

### Repeated activation of the same functions

To ensure safe operation via infrared, a forced pause of 2 seconds is inserted between repeated activations of the same function.

In modern infrared remote controls, this pause is not required, as repeated commands are distinguished by a so-called „toggle bit“ in the signal.

If a new **4** function button is pressed, the selected function is executed immediately.

---

## Different Device Addresses (Infrared Operation)

If several beds with a **BIF8+IE** are used in one room, it must be ensured that each bed only responds to its corresponding remote control. For this reason, the IR reception address can be set **by a trained technician**. Each **BIF8+IE** can be assigned one of four possible IR device addresses, allowing up to four beds controlled by **BIF8+IE** to be used in one room.

## Monitoring of Switching Contacts

The **BIF8+IE** detects if one of the switching contacts no longer disconnects electrically due to a defect. In this case, the enabling of the switching contacts is deactivated. This faulty condition is indicated by the flashing red ② indicator light.

## System Self-Monitoring

Safety measures have been implemented in the **BIF8+IE** for a possible malfunction of the control unit. In the event of a failure, the enabling of the switching contacts is deactivated.

---

## 9. Settings (only by trained specialist personnel)

### Operating Modes

There are three operating modes for the Handset HS plus hand control unit:

- IR operating mode (default setting): reception of external infrared signals
- Easywave operating mode: reception of external Easywave radio signals
- Pure button operation mode: no reception of external signals

The modes are exclusive, i.e. only one external source can be selected at a time.

### Change Address Range

If several **BIF8+IE** units are operated in one room, it is recommended to change the address range of the **BIF8+IE** when using external infrared remote controls.

The change can be made directly on the hand control unit. The change can also be made using the optional IR handheld remote control (CSS Art. No. 20324).

**Note:** If the address range is changed via the hand control unit, it must subsequently also be adjusted accordingly on an existing IR remote control.

### Programming and Settings

Setting the operating mode, pairing external radio transmitters and learning external infrared remote controls are carried out according to the following tables.

Resetting to factory settings is also described in the tables.

### General Notes on Programming

- **Acoustic signals:** Short signals indicate status or mode entry/exit; longer sounds confirm successful actions.
  - **Automatic exit:** Most programming modes can also be exited by waiting (timeout).
-

## Display current operating mode

Function Area	Button Action	Duration/ Frequency	Status LEDs ( <i>MODE/OPT</i> )	Function/Result
Query status	MODE button	Brief press	<p>MODE button lights up <b>red: IR source</b> active (default)</p> <p>MODE button briefly lights up <b>green: Easywave source</b> active</p> <p>OPT button briefly lights up <b>green: pure button operation</b>, no reception of external signals</p>	Displays the currently selected wireless source

## Change operating mode

Function Area	Button Action	Duration/ Frequency	Status LEDs ( <i>MODE/OPT</i> )	Function/Result
Change source (preview)	MODE button	Double press (within $\leq 350$ ms)	MODE button (red/green) or OPT button (green) <b>flashes</b> depending on the operating mode	Switches to preview of the next source in sequence: Button operation -> IR -> Easywave
Change source (confirm)	OPT button	Press and hold (~3 s) during preview	<b>Short confirmation sound</b> ; LED lights <b>continuously</b> for 2 s in the selected color	Confirms the selected source in preview mode

## Pairing Easywave Transmitter

Step	Objective	Button Action	LED Feedback (OPT button)
<b>1. Preset</b>	Ensure that the source is Easywave	Press MODE button briefly	MODE button must pulse green
<b>2. Preparation</b>	Prepare to start Easywave programming	<b>Press OPT button briefly</b>	OPT button <b>lights up green continuously</b> (~5 s time window)
<b>3. Programming mode</b>	Enter Easywave programming mode	<b>Hold OPT button</b> (~3 s within the 5 s window)	OPT button <b>flashes green</b>
<b>4. Select function</b>	Select memory position for the transmitter	<b>Press and hold movement button 1 to 10</b>	Selected <b>movement button flashes green</b> (waiting state)
<b>5. Pair transmitter</b>	Trigger the transmission command	Press desired button / trigger command on external <b>Easywave transmitter</b>	Selected movement button <b>lights up green for 2 s</b> (confirmation)
<i>Optional</i>	Steps 3 and 4 can now be repeated for additional transmitters		
<b>6. Exit</b>	Exit programming mode	<b>Press MODE or OPT button</b> briefly or wait approx. 60 s	Return to normal operating mode (MODE lights up green briefly)

## Train Infrared Transmitter

In this case, the hand control unit acts as a transmitter to transfer the IR codes to the environmental control or communication device. The IR signals are emitted at the top of the hand control unit **7**.

Step	Objective	Button Action	LED Feedback ( <i>OPT button</i> )	Action
<b>1. Preset</b>	Ensure that the source is IR	Press MODE button briefly	MODE button <b>must pulse red</b>	The hand control unit is set for IR operation
<b>2. Preparation</b>	Prepare to start IR programming	<b>Press OPT button briefly</b>	MODE button <b>lights up red continuously</b> (~5 s time window)	Step 3 must be carried out within this time window
<b>3. Enter mode</b>	Enter IR programming mode	<b>Press and hold OPT button (3 s)</b>	OPT button <b>flashes red</b>	Entry into IR programming mode
<b>4. Send IR command</b>	Send the desired IR command	<b>Press/hold</b> one of the movement <b>buttons 1–10</b>	Corresponding <b>button LED</b> lights up	Sends via IR with the current address range. Multiple buttons are allowed in one session
<b>5. Exit</b>	Exit programming mode	<b>Press MODE or OPT button</b> briefly	Return to normal operating mode	Ends IR programming
<b>6. Timeout</b>	Exit programming mode due to inactivity	Wait approx 60 s (idle)	Return to normal operating mode	Ends IR programming

## Change Infrared Address Range

Step	Objective	Button Action	LED Feedback (OPT button)	Value
<b>1. Enter change mode</b>	Enter address range selection mode.	<b>Hold OPT button (&gt;4 s)</b>	<b>OPT lights continuously green; buttons 1, 2, 3, 4 flash very quickly</b>	–
<b>2. Select address range</b>	Select desired address range	<b>Briefly press one of the following buttons: 1, 2, 3 or 4</b>	Selected button <b>flashes green quickly</b>	<b>Button 1: Range 1 Button 2: Range 2 Button 3: Range 3 Button 4: Range 4</b>
<b>3. Confirm &amp; save</b>	Save the new address range	<b>Press MODE button briefly</b>	OPT lights up briefly and the new range is saved	–
<b>4. Cancel</b>	Exit change mode without saving (alternative to step 3)	<b>Press OPT button briefly</b>	Return to normal operating mode	No change saved
<b>5. Timeout</b>	Exit change mode without saving (alternative to step 3/4)	Wait approx. 10 s	Return to normal operating mode	No change saved

## Factory Reset

- Sets operating mode to IR
- Deletes all stored Easywave transmitters
- Resets address range to "1"
- Restarts the hand control unit

Function Area	Button Action	Duration/ Frequency	Status LEDs (MODE/OPT)	Function / Result
Factory reset	<b>MODE button</b> + <b>OPT button</b>	Press and hold simultaneously (>5 s)	MODE button (red / green) or OPT button (green) <b>flashes</b> depending on the operating mode	All settings are reset (delivery condition)

## Special Function – Trigger Reference Run

Some nursing beds require a reference run. By pressing buttons 9 and 10 simultaneously, a special function is triggered in the **BIF8+IE**. The correct relays may then need to be assigned in the **BIF8+IE** via software.

In the **BIF8+IE**, the subsequent commands from the hand control unit are then passed through without time limitation until 3 minutes have elapsed (i.e. the auto-stop function is blocked).

**During this time, the bed must not be used for its normal function for safety reasons, meaning that no patient may be in the bed.**


The 3-minute duration is additionally indicated by a flashing red LED (OPT button)

## Individual Settings with BIF Software (this software is only provided to trained specialist personnel)

First, supply the **BIF8+IE** with power by connecting the power supply unit.

Connect the **BIF8+IE** to a notebook/PC: Remove the side housing cover on the control unit and connect it using a USB mini cable (not included in the scope of delivery). Then start the Universal Configurator software.

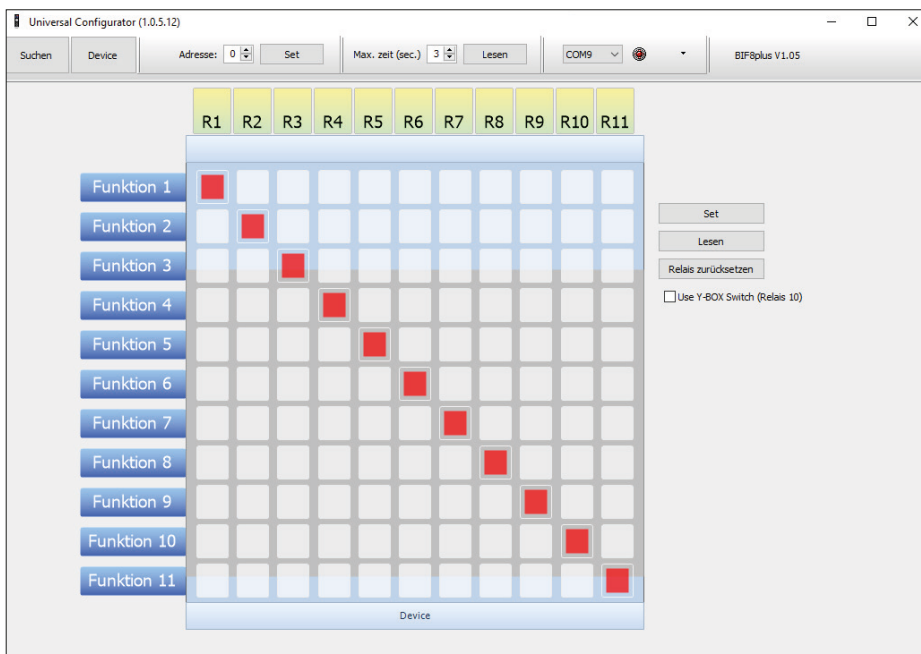Functions can now be assigned to the corresponding relays by clicking.

## Error Diagnosis – Enabling & Switching Contacts

The **BIF8+IE** detects if one of the switching contacts no longer disconnects electrically due to a defect. In this case, the enabling of the switching contacts is deactivated.

For the safety of the user, further switching commands are ignored.

This faulty condition is indicated by **the red LED** on the front side of the housing.

## 10. Maintenance / Review

The device must be checked for safety and proper function at **regular intervals of 3 months**. The inspection must be carried out by **trained specialist** personnel. The following must be checked in detail:

- Functional test
- Connectors and cables for damage and contamination
- Mounting parts for secure fastening and damage

## 11. Support

For all technical questions regarding the product, please first contact your specialist dealer from whom you purchased this product. If necessary, if the specialist dealer is not available or cannot assist, please send us an email at [support@csslabs.de](mailto:support@csslabs.de).

Please be sure to include the error or operating problem and the serial number.

## 12. Technical specifications

### Control unit:

Dimensions: 145 × 106 × 45 mm

Weight: approx. 700 g

IP protection class: IP65

### Power supply:

power supply unit 100–240 V AC, stabilized 12 V DC / 700 mA; complies with IEC 60601-1

Switching contacts: 48 V DC / 500 mA; 11 functions, 1 enabling contact

Infrared coding: 38 kHz, RC5 code

### Radio (Easywave):

868.3 MHz (Eldat Easywave), receive-only (RX) for external radio signals

---

**Hand control unit:**

Dimensions: 168 × 75 × 36 mm

Weight: approx. 150 g

**Temperature ranges**

Operation: 0 °C to 45 °C

Transport and storage: -10 °C to 50 °C

### 13. Important information for system manufacturers

The specialized dealer (system manufacturer) must independently verify whether the system consisting of the bed and the special control unit meets the required safety and performance standards.

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
The installing specialized dealer (system manufacturer) is responsible for properly documenting the combination of the **BIF8+IE** with the control system of the care bed. This documentation must, in particular, include which bed was connected, how the installation was carried out, and the date of installation.

### 14. Environmental protection and disposal

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
All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points. Please dispose used batteries and rechargeable batteries in a recycling bin for used batteries or via special resellers.

WEEE DE94635635

**ASSISTIVE TOOLS FOR A BETTER LIFE**

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© 2026 CSS MicroSystems GmbH