Bedienelemente / Control elements

Technische Daten / Technical specifications

Stromversorgung / Power supply:

USB Port AMAneo MX

Schnittstelle / Interface:

Bluetooth® Classic

Anschluss / Connection:

USB A

Maße / Dimensions:

59,5 × 48,5 × 17 mm

Gewicht / Weight:

31 g

WEEE DE94635635

© 2026 CSS MicroSystems GmbH

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#20544 WheelChair-Link für AMAneoMX V1.0 DE

Instructions for Use:

#20544 WheelChair-Link for AMAneoMX V1.0 EN

Support: support@csslabs.de

#16470 V1.0 DE/EN

DE/EN (EU)

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INSTRUCTIONS
FOR USE

WheelChair-Link for AMAneo MX

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The WheelChair-Link allows electric wheelchairs with a built-in Bluetooth® interface to be connected to the AMAneo MX for the joystick's mouse function, enabling the wheelchair joystick to be used to control gaming consoles and similar devices.

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Setup/Use

Plug the WheelChair-Link into the AMAneo MX (*both USB ports*). It will then automatically register there.

In normal mode, the WheelChair-Link does not actively search for a connection but only reconnects to the last known wheelchair.

To pair with the wheelchair, press the pairing button **2**. This activates the search for a wheelchair for approx. 30 seconds; the LED **1** flashes blue.

Put the wheelchair into pairing mode (*see the wheelchair controller's instructions*).

As soon as the connection is established, the search ends automatically and the device saves this input device for future reconnections. The LED **1** lights up green.

Pressing the button briefly again manually ends the search.

LED Colors:

Flashing blue: Search/pairing mode active

Green: Wheelchair connected

Red: No wheelchair connected

Notes:

The system always prioritizes reconnecting to the most recently paired or most recently known wheelchair. If no wheelchair is connected, the LED **1** remains red. After the search time has elapsed, the search automatically turns off.