Bedienelemente/Control elements

Technische Daten/Technical specifications

Stromversorgung/Power supply:
Über AMAneo MX/Via AMAneo MX

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The WheelChair-Link allows electric wheelchairs with a built-in Bluetooth® interface to be connected to the AMAneo MX for the joystick's mouse function, enabling the wheelchair joystick to be used to control gaming consoles and similar devices.

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Setup/Use

Plug the WheelChair-Link into the AMAneo MX (both USB ports). It will then automatically register there.

In normal mode, the WheelChair-Link does not actively search for a connection but only reconnects to the last known wheelchair.

To pair with the wheelchair, press the pairing button ❷. This activates the search for a wheelchair for approx. 30 seconds; the LED ❶ flashes blue.

Put the wheelchair into pairing mode (see the wheelchair controller's instructions).

As soon as the connection is established, the search ends automatically and the device saves this input device for future reconnections. The LED ❶ lights up green.

Pressing the button briefly again manually ends the search.

LED Colors:

Flashing blue: Search/pairing mode active

Green: Wheelchair connected

Red: No wheelchair connected

Notes:

The system always prioritizes reconnecting to the most recently paired or most recently known wheelchair. If no wheelchair is connected, the LED ❶ remains red. After the search time has elapsed, the search automatically turns off.

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© 2026 CSS MicroSystems GmbH