DE/EN

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**INSTRUCTIONS
FOR USE**

ClickMan

C-SS
MICRO SYSTEMS

DE Seite 3-41

EN Page 42-79

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WEEE DE94635635

Table of Contents

1. Introduction	44
2. Intended use	44
3. Explanation of symbols	45
4. Safety instructions	46
5. Cleaning and disinfection	48
6. Scope of delivery	49
7. Device Buttons, Ports, Display	50
8. Use of the device	52
Installation:	53
Use default setup:	54
Change ClickMan setup:	54
Multi-Mode Betrieb:	55
② <i>Switching behavior:</i>	55
③ <i>Anti tremor filter:</i>	57
<i>Switching function:</i>	58
<i>Feedback:</i>	60
<i>Output jack:</i>	61

Use of Multi-Mode:	61
Single Action Mode:	61
Change Mode	62
The Mode Change Click	62
9. Settings	64
Hardware description:	64
Configuration of Output 1	68
ClickMan Configurator Software	69
② Sensor monitoring	71
③ Anti-Tremor	72
④ Multi-Mode	72
⑤ Standard mode	73
⑥ Multi Modes.	73
Toolbar.	74
10. Maintenance / Review	76
11. Support	76
12. Technical specifications	77
13. Conformity / Standards	77
14. Environmental protection & disposal	78

1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

2. Intended use

ClickMan is an adaptive, universal sensor controller. Buttons and sensors connected to the input (**AssistX Port**) can trigger a wide range of configurable actions at the outputs.

ClickMan serves in particular to implement individual user requirements. For example, short-time actuation can be ignored or repeated accidental actuation can be avoided to ensure safe functioning. It is also possible to control two devices with one sensor/button.

The patented **AssistX Port** (input) automatically recognizes which types of control elements are connected and can also supply them with power (e.g. **AssistX AS one** and **AssistX ProX one**).

ClickMan serves a **AssistX Port** sensor input, and two potential-free outputs. One of the outputs can be used as a normal open contact.

A feedback can be taken optical or acoustical for either the input or the output.

With the integrated lithium polymer battery, the device is also suitable for mobile applications.

The firmware can be conveniently updated with the **ClickMan** configurator. Furthermore, the settings of the **ClickMan** can be protected by a pin code.

3. Explanation of symbols

 Safety instructions!

 Important instructions!

4. Safety instructions

Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification. The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personal.

Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting.

The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

Repair

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

Interaction with medical devices

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using other active medical devices. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

5. Cleaning and disinfection

Before cleaning the device, switch it off and disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

6. Scope of delivery

- **ClickMan**
- USB cable
- USB power adapter
- mono jack plug
- stereo jack plug

7. Device Buttons, Ports, Display

-
- | | | | |
|---|---------------|---|-------------------------------|
| 1 | Output 1 | 4 | On and Off switch |
| 2 | Output 2 | 5 | Input (AssistX Port) |
| 3 | Mini USB port | 6 | LED & battery status |
-

1 Output:

- The output contact can be connected to either of two 3.5 mm jack sockets. Both mono and stereo jack plugs can also be used.
- Output 1: stereo jack plug, 3,5mm, floating, ring and tip contact may be used as normally open and normally closed contact (suitable for call systems).
- Output 2: stereo jack plug, 3,5mm, floating, normally open.
- Different output switchmodes: Toggle switch, Trigger, Impuls, Repeated pulse, Configurable „doubleclick“ output and output may be switch off automatically even if inputsensor is still active.

4 Input AssistX Port sensor connection:

- Buttons and single sensors with opening or closing contact.
 - ProX proximity sensor
 - Active sensors from the AssistX family (**AS one**, **IR one**, **RX one**)
 - Customizable software filter for the input signal (antitremor filter)
-

6 Battery charge level

Press the Button for 2 sec.

 red:	< 25%
 orange:	> 25%
 light green:	> 50%
 green:	> 75%

8. Use of the device

ClickMan is configured to switch **1** output 1 as long as the input sensor is actuated. If this functionality is sufficient for your application, all you have to do after unpacking is set the **2** battery switch on the circuit board to **[ON]** (see page 27). Then you can start directly with the **standard configuration**. If the **ClickMan** is still not working, fully charge the battery.

If you want to use more functions, you have to use the ClickMan Configurator software (see page 32). In this case, start with **change ClickMan setup** (see page 16).

Installation:

Open the case and set the ² battery switch to **[ON]**. The printed circuit board is accessible after opening the housing. To open it, the cover on each narrow side of the housing must be unlocked. This is done by pressing the snap strips on the left and right inwards with a small slotted screwdriver, turning them slightly and lifting the lid in the process.

! The housing may only be opened by an authorized experts.

Use default setup:

Using the default setup, the one and only function of the **ClickMan** is to switch Output 1 as long as the input sensor is activated. Close the housing. Now connect the input sensor to **4** socket S. The switching signal can be tapped off with a 3.5 mm mono or stereo jack plug at **1** output socket 1. Configure the jack socket using **9. Settings** (see page 32).

Change ClickMan setup:

If you would like to configure the other options of our **ClickMan**, you can use our **ClickMan Configurator Software**, which you can download from www.csslabs.de/downloads.

After the device is opened, put the **2** battery switch in the **[ON]** position. The switching signal can be tapped off with a 3.5 mm mono or stereo jack plug at **1** output socket 1. Configure the jack socket with **9. Settings** (see page 32). Close the device. Plug the input sensor into the **4** socket. Start the **ClickMan** Configurator as described in the settings. Connect **ClickMan** to the PC with a USB cable.

Multi-Mode Betrieb:

ClickMan can save up to five different switching behaviors. In addition to the standard mode, up to four other working modes can be saved. The mode can be changed with an individual long or double click. Each mode can be assigned its own color to differentiate between the modes and for easier orientation.

Settings for **sensor monitoring** and for the **anti tremor filter** apply overridingly to all configured operating modes.

Switching behavior, **feedback** and **output socket** can be adjusted as required for each of the five modes.

② **Switching behavior:**

If the sensor is unplugged or the sensor cable is defective, ① output 1 a configurable switching function can be generated. The prerequisite for this is that a sensor with an opening contact is used. (e.g. with proximity sensor ProX or Kajo AssistX Button). For technical reasons, cable break detection is not possible with sensors with a **normally open** function. An unplugged sensor, on the other hand, is recognized for every sensor type. In addition, the sensor error can also be indicated optically and acoustically.

For technical reasons, continuous operation of a sensor with an **normally closed contact** has the same effect as a cable break on the sensor. If the sensor is pressed for longer than the set triggering time, this also triggers a switching process for opening sensor types.

! If you want to enable permanent actuation of an opening sensor, you can switch off sensor monitoring. However, this means that a defective sensor cable or a missing sensor is no longer detected.

- **Alarm symbol**  Sensor monitoring on. Settings for feedback, triggering time and switching behavior are taken into account.
 - **Alarm symbol**  Sensor monitoring off. Output 1 is not switched.
 - **Feedback**   Visually and/or acoustically indicates a sensor error.
 - **Trigger timer 10/60 sec**
Time between the occurrence of a sensor error and the triggering of a switching signal.
-

- ***Impuls 100 - 5000 ms***

A switching pulse is generated at ① output 1. The duration can be set between 100 ms and 5 sec.

- ***Continuous signal***

In the event of a sensor error, ① output 1 is permanently switched.

③ ***Anti tremor filter:***

The ③ anti tremor filter enables people to generate clear switching instructions despite strong tremors. The device is delayed in its reaction to the input sensor by the set times.

- ***Min. holdtime 0 - 1500 ms***

If the sensor is actuated and released again before the entered set hold time is reached, the actuation is discarded.

- ***Min. releasetime 0 - 1,5 sec***

An actuated input sensor is only recognized as released, if it remains released for at least the set time.

Switching function:

The switching behavior can be set as follows for each mode:

- ***Switch***

The selected output is switched, taking into account the input filter, until the input sensor is released again.

- ***Long hold breaks***

If the user cannot safely release an actuation, **ClickMan** can automatically end the switching process after the set time, even if the input sensor is still actuated.

- ***Toggle Switch***

The selected output is activated each time the input sensor switched on and off alternately.

- ***Impulse (100 ms - 5000 ms)***

The selected output is switched for the set pulse duration, after which it returns to the idle state.

- ***Repeated pulses***

A repeating pulse followed by a pause is generated at the selected output.

- ***Time of impulse (100 ms - 5000 ms)***
Switch-on time for the first and for the repeated switching pulses.
 - ***Delay of impulse***
Pause before the start of the repeated switching pulses.
 - ***Repetition rate***
Time from the start of a switching pulse to the start of the next.
 - ***Doubleclick***
Two pulses are output at the selected output when the input sensor is actuated. The preset values roughly correspond to the usual doubleclick with a PC mouse.
 - ***Time***
Duration of the first click.
 - ***Delay***
Pause until the second click.
-

Feedback:

The feedback of input sensor and output can be displayed optically and acoustically. The feedback can be set for each mode separately. It is advisable to assign a individual color to several modes in use.

- **Output**

The output switching status of the selected output is displayed. (LED color / acoustic signal generator)

- **Input**

The switching state of the input sensor is displayed. The set input filters are ignored here. (LED color / acoustic signal generator)

- **LED**

The respective mode is represented by the feedback color. The brightness can vary between 0-5, where 0 means off.

- **Sound**

The sound can vary between 0-10, where 0 means off.

- **Mode Change Click**

The mode change click is a hard-coded acoustic one signal and is activated by a double or long click on the input sensor caused.

Output jack:

The contacts of the associated switching relays can be tapped at the 3.5 mm stereo jack sockets ① and ②. Both mono and stereo jack plugs can be used. If a mono plug is used at ① output 1, the ring contact must be separated from the circuit (see page 31).

- **Output 1 with changeover relay ①**

The **normally closed** and the **normally open** contact of the changeover relay can be tapped at the ring and tip contact of the 3.5 mm stereo jack socket.

- **Output 2 with solid state relay ②**

The **normally open** contact of the semiconductor relay can be tapped at the tip contact of the 3.5 mm stereo jack socket.

Use of Multi-Mode:

There are two ways to use the expansion modes:

Single Action Mode:

The device works in ⑤ standard mode. This can be configured in any

way. If a regular click is made, the standard mode is executed.

Change Mode

After a modechange click you get to mode 1. Mode 1 can also be configured in any way. After another regular click, the mode is executed once. When the action from this mode is complete, the **ClickMan** automatically switches back to the ⑤ standard mode. If you would like to run another of the four modes, you can access the individual modes by clicking on the mode change button.

The Mode Change Click

The expansion modes are selected by double-clicking or long-clicking on the input sensor.

- ***Doubleclick***

The doubleclick speed is set with the ClickMan configurator. A doubleclick similar to that of a PC mouse is used as the default value. The color of the new mode is briefly displayed.

- ***Longclick***

A click becomes a long click when the set hold time is reached. Before

reaching the long click, the LED flashes in the color of the next mode. If the input sensor remains activated, the set multi-mode action is carried out. In the regular long-click, the switching function of the current mode is initially suppressed until the long-click time has been reached (only applies to toggle, impulse and double-click functions). If the switching function is to be triggered directly when the input sensor is actuated, select the ④ **"Transparent click"** extra for the long click.

1 **LiPo Battery**

Reverse polarity protected connection for the lithium polymer battery.

2 **Battery ON/OFF**

Isolates battery voltage from circuitry during storage and shipping. After you set the slide switch to **[ON]**, it remains in this position.

3 **USB socket**

Charging the battery with a standard USB charging adapter. Providing the supply voltage. Data transfer when configuring the **ClickMan** on the PC.

4 **AssistX Port**

Connection socket for the input sensor. If the **ClickMan** detects an active sensor (**AssistX ProX one**, **AssistX** sensors), it automatically supplies the sensor. Other sensors or buttons can also be plugged into this socket. These can be connected with both a mono and a stereo jack plug. **ClickMan** regularly checks the input socket and automatically adapts to the sensor used.

5 **Switching Output 1**

3.5 mm stereo jack socket, potential-free. The contacts can be connected to the opening and closing contact of the changeover relay. The common contact of the relay is at the shaft contact of the socket. Use the slide switches 7 and 8 to configure the socket for the jack plug used and your requirements. This connection is the right one if the **ClickMan** is to switch as an opener at the output and the current to be switched is not greater than 170mA.

6 **Switching Output 2**

3.5 mm stereo jack socket, potential-free. The tip contact of the socket is connected to the closing contact of the solid state relay. In the switched state, the tip contact is connected to the shaft contact. This connection is the right one if, for example, an environment control device is to be controlled. Currents of up to 500 mA at max. 48V can be switched. The 1 output requires less power than output 1. The battery life is therefore increased if your application is connected to this output.

7 **SW1 - Set 1 Output 1 tip contact**

Connects the NC/NO contact of the changeover relay to the tip contact of the 3.5 mm jack socket. The common contact is at the shaft contact.

8 **SW2 - set 1 output 1 ring contact**

Connects the ring contact of jack socket 1 to the break contact of the relay, or disconnects it from the circuit. If a mono plug is used, the ring contact must be separated from it.

9 **Power off and state of charge**

Press the button for 2 seconds. The 6 LED shows the charge status of the battery. If you keep pressing the button, the 6 LED goes out and a beep sounds. The device is now switched off and in energy-saving mode. Only the internal system monitoring is still active. An actuation of the input sensor is no longer detected. To establish a connection with the **ClickMan** configuration software, the device must be switched on. Switch on the device by pressing the button again for 2 seconds. The 6 LED lights up **green** to confirm, then two beeps sound again. The device is now ready. If the battery switch is set to **[ON]** or the USB cable is plugged in, the device is automatically on. When switched on, the color of the current mode is briefly displayed on the 6 LED approx. every 5 sec.

10 **External RGB-LED**

Terminal for an external RGB LED module.

Configuration of Output 1

Overview settings for ❶ Output 1:

Tip contact ❸ SW1 with pos1 as opener and pos2 as closer.

Ring contact ❷ SW2 with Pos1 as opener, Pos2 not connected.

Shaft contact as common contact (tongue) of the relay.

Output 1	❸ SW2	❷ SW1	❶ SW1
<p>3,5 mm mono jack plug Normally closed or normally open contact.</p>	 <p>Cut connection to circuit!</p>	 <p>NC contact at tip</p>	 <p>NO contact at tip</p>
<p>3,5 mm stereo jack plug Normally closed and normally open contact.</p>	 <p>NC contact at ring contact</p>		 <p>NO contact at ring contact</p>

ClickMan Configurator Software

The various setting options of the **ClickMan** are made on the PC with the configuration software. You can download and install this from www.csslabs.de/downloads.

! Please start the configuration software first and only then connect the **ClickMan**!

Start the **ClickMan** Configurator. Now connect **ClickMan** to the PC using the supplied USB cable. After the USB cable has been plugged in, the **ClickMan** is switched on automatically and reports to the software with its current version number. The configuration software ① automatically establishes the correct connection to the **ClickMan**.

The **ClickMan** can be configured to sound an alarm if no sensor is plugged in. In this case, also plug a sensor into the socket to avoid an alarm tone.

First set the general settings. The ② **sensor monitoring** and the ③ **anti tremor** settings apply to each configured mode.

You can tell from the color of the symbols whether the associated setting is activated. If the icon is grayed out, it is disabled:

activated: 

deactivated: 

② **Sensor monitoring**

Following can be set in detail:

- **Sensor monitoring (de)activate**

If you deactivate the alarm symbol, no switching operation will be triggered, even if the sensor (NC type) is actuated continuously. A defective cable or an unplugged sensor will not be recognized.

- **Trigger time**

The time from noticing a sensor error to triggering the switching process can be set.

- **LED / Sound**

Depending on the setting, the ⑥ LED flashes **red** in the event of a sensor error and/or a recurring beep is generated, the volume of which can be adjusted.

- **Impulse**

In the event of a sensor error, switching ① output 1 is switched on for the set switching time and then switched off again.

- **Permanent switch**

In the event of a sensor error, switching ① output 1 is switched on.

③ **Anti-Tremor**

Following can be set in detail:

- **Min. hold time 0-1500 ms**
The input is ignored until the set time is reached.
- **Min. release time 0-1500 ms**
Release of the input sensor is ignored until the minimum release time is reached.

④ **Multi-Mode**

Following can be set in detail:

- **Single Mode & Change Mode**
Mode 1 is for a one-time action. Mode 1-Mode 4 can be added and configured in addition to the standard mode.
 - **Mode Change Click**
Long click (Time & Transparency), Double Click (Time & Delay)
-

⑤ **Standard mode**

Create the functioning of the standard mode:

- **Output**

Choose between ① Output 1 and ② Output 2. Under Function, set the desired switching function. Under the „**button**“ function, you can also select an automatic switch-off of the process and adjust the time. If you don't want this, set the value to **0**.

- **Feedback**

Select whether the switching state of the output or the input should be reported. You can also set the ⑥ LED color and brightness here. If you use several modes, it makes sense to assign each mode its own color. You can also adjust the sound, including the volume, to suit you. There is an acoustic feedback when the input sensor is activated or the output is switched on.

⑥ **Multi Modes**

A **[+]** will now appear next to the ⑤ Standard Mode tab. Four more modes can be configured in this way. You can delete a mode again with ⑥ **[x]** in the upper right corner of the mode tab.

Toolbar

- **Write to ClickMan** ⑦

Transmit the settings.

- **Read from ClickMan** ⑧

Show and read the **ClickMan** settings.

- **Battery status** ⑨

Current battery charge.

- **Reset parameters** ⑩

Reset **ClickMan** including pin code.

- **Manage Setups** ⑪

Saving and reloading various settings. To save the new settings, click ⑬ „**Save Setup**“. You will find the saved settings in the setup management window under the name you enter.

- **Advanced** (12)

Here you can (re)assign or change your secret number (PIN) for protection. If you no longer wish to use a PIN, enter **"0000"** as the new number. You can also update the firmware, including the latest configuration software: With every current version of the **ClickMan** configurator, you will receive the latest firmware for **ClickMan**. After clicking „Update Firmware“, the existing firmware will be overwritten with the new firmware.

10. Maintenance / Review

The following checks must be carried out every three months:

- Function
- Connector for damage and dirt
- Damage of the housing and other components

11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to support@csslabs.de. Please be sure to include a description of the problem and the serial number of the device.

12. Technical specifications

Dimensions: 102 x 54 x 30 mm

Weight: 95 g

IP Rating: IP 54

Power supply: LiPo Battery, 3.7 V, 1250 mAh, USB socket, mini USB, 5V DC, max 500 mA

Switching Contacts: semiconductor relay, changeover contact (S1) max. load capacity 170 mA, max. 48 VDC; semiconductor relay (S2) max. load 500 mA, max. 48 VDC

Temperature ranges

Use: 0 °C to 45 °C

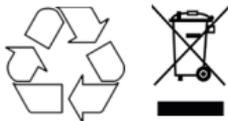
Transportation and storage: -10 °C to 50 °C

13. Conformity / Standards

 This product is CE compliant.

14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

THE PASSIONATE ASSISTIVE TECHNOLOGY MANUFACTURER

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© 2022 CSS MicroSystems GmbH