DE/EN (EU)

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**INSTRUCTIONS
FOR USE**

AssistX ProX One

C.S.S.
MICRO SYSTEMS

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WEEE-DE94055635

Table of Contents

1. Introduction	13
2. Intended use	13
3. Explanation of symbols	13
4. Safety instructions	14
5. Cleaning and disinfection	15
6. Scope of delivery	16
7. Device Buttons, Ports, Display	17
8. Use of the device	18
9. Settings	19
10. Maintenance / Review	19
11. Support	19
12. Technical specifications	20
13. Conformity/Standards	20
14. Environmental protection and disposal.	21

1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

2. Intended use

AssistX ProX one is a proximity sensor that can be triggered without contact with moving parts of the body. It is intended in particular for the use of triggering calls with **AssistX Mobil** or **AssistX Call**. It can also be used in conjunction with ClickMan® to control other devices. **AssistX ProX one** can be flexibly positioned using the supplied floor stand or the supplied mounting arm.

3. Explanation of symbols

Safety instructions!

Important instructions!

4. Safety instructions

Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification.

The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

Repair

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

Interaction with medical devices

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using other active medical devices. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

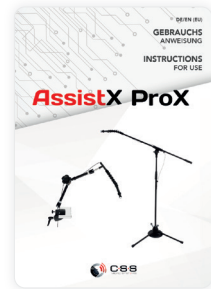
5. Cleaning and disinfection

Before cleaning the device disconnect all cables. The device should be cleaned daily with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

6. Scope of delivery

- #20454 Tripod consisting of stand, weight, flexible arm with sensor
- #20455 Flexible mounting arm with integrated sensor and clamp
- Instructions for Use

7. Device Buttons, Ports, Display

EN

#20454 tripod

1 Proximity sensor

2 Tripot with weight

#20455 mounting arm

1 Proximity sensor

3 Clamp

2 Flexible mounting arm

8. Use of the device

The proximity sensor is plugged into the **AssistX Port** of **AssistX Call**, **AssistX Mobil** or Clickman via a 3.5 mm jack plug and can then be used like a conventional single button sensor. It is triggered by approaching the sensor (0-5 mm). The sensor is to be positioned with the respective mounting system in such a way that it has sufficient distance in the inactive state, but can be safely actuated when triggered. The release can be done with the head, limbs and hands.

With the mounting arm, all three joints are tightened or loosened at once with the central rotary knob.

Preparations on the tripod (#20454):

For mounting on the tripod, the tripod must first be set up in a stable manner. Fix the weight with the enclosed locking pin. Pull the stand tube all the way up until you can see the hole for the cotter pin. Then insert the cotter pin to secure the weight. In addition, the boom is screwed onto the stand tube.

Preparations on the mounting arm (#20455):

To assemble the mounting arm, first insert the mounting arm with the connection piece with the **smaller** inner thread (1/4") into the base clamp. Press the locking pin on the base clamp and then secure the connection piece with the screw (note the flattened side!). Screw in the flexible arm with the sensor at the other end of the mounting arm (larger inner thread, 3/8") and counter with the threaded washer.

9. Settings

No settings can be made on the **AssistX ProX one**. Correct positioning sets the correct distance between the sensor and body part.

Take care when positioning: Make sure that no injuries are possible with involuntary movements! Pay attention to the eyes in the head area!

10. Maintenance / Review

The device usually does not require any maintenance. We recommend however, check the following **every 3 months**:

- Functional test
- Connector and cable for damage and contamination
- Mounting parts for secure holding function and damage

11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to support@csslabs.de. Please be sure to include a description of the problem and the serial number of the device.

12. Technical specifications

Dimensions:

Tripod: height 1040 - 1625 mm, boom length max. 1030 mm

Mounting arm: 360 - 885 mm

Sensor: $\varnothing$ 12 mm

Weight: 5,1 kg (tripod); 1,7 kg (mounting arm)

IP Rating: IP 54

Power supply: 12 V DC through AssistX Port

Connector: 3,5 mm jack plug

Switching distance: ca. 0-5 mm

Temperature ranges

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

13. Conformity/Standards

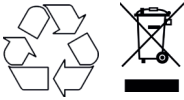
 This product is CE compliant.

AssistX ProX one meets the requirements of DIN VDE 0834 in combination with **AssistX Mobil** and **AssistX Call**.

14. Environmental protection and disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points. Please dispose used batteries and rechargeable batteries in a recycling bin for used batteries or via special resellers.

EN

WEEE DE94635635

THE PASSIONATE ASSISTIVE TECHNOLOGY MANUFACTURER

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© 2021 CSS MicroSystems GmbH