DE/EN

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**INSTRUCTIONS  
FOR USE**

# RemoBIF BM

**CSS**  
MICRO SYSTEMS

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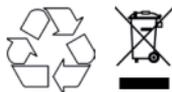
---

## 1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

## 2. Intended use

RemoBIF BM is an MDR-compliant bed control interface that enables safe control of your own bed with individual control elements, communication and environmental control devices. The RemoBIF BM is specially designed for Burmeier's „Dali“ care bed, which uses protected Bluetooth® control. The manufacturer Burmeier has expressly approved the combination of the „Dali“ bed with the RemoBIF. These beds can be safely controlled with infrared signals or Easywave radio. Both are supported by many common communication and environment control devices and make the choice of control elements such as proximity or acoustic sensors, buttons, head and eye control flexible.

Safety instructions!

Important instructions!

---

## 4. Safety instructions

### Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification.

The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personal.

### Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting. The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

---

## **Repair**

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

## **Interaction with medical devices**

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using active medical devices. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

---

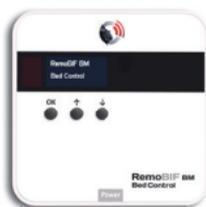
## 5. Cleaning and disinfection

Before cleaning the device, disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.


**Do not use sprays** that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

## 6. Scope of delivery


- **RemoBIF BM**
- USB-C power adapter
- Wall mount (optional)
- Magnet for handset
- Instructions for Use

## 7. Device Buttons, Ports, Display

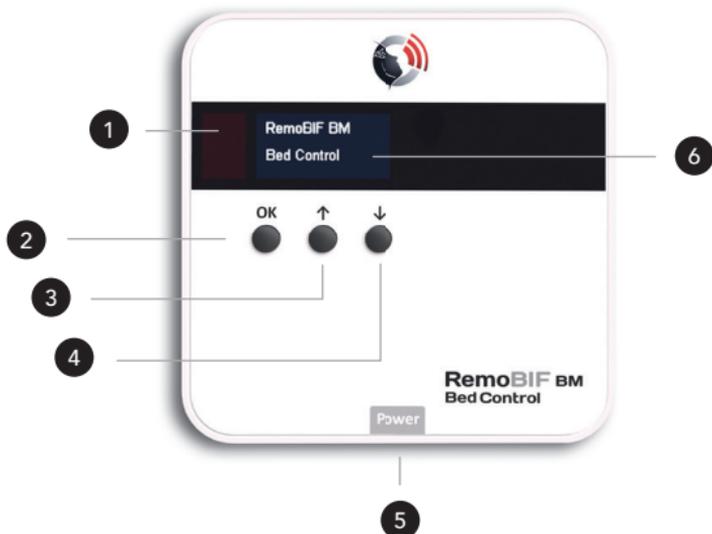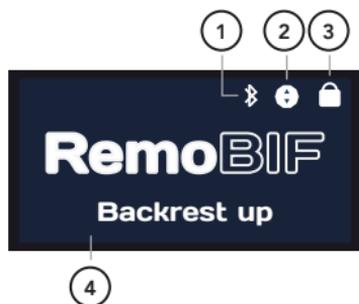- 1 Infrared sensor/transmitter
- 2 Confirmation button [OK]
- 3 Navigation button [↑]
- 4 Navigations button [↓]
- 5 USB-C power adapter
- 6 Display

- 1 Bluetooth connection to the bed (appears only when pressed)
- 2 Bed actuator moves
- 3 Current function blocked
- 4 Plain text: recognizing a function (IR or Easywave)

## 8. Installation

Before use, the bed must be connected with **RemoBIF BM** once.

### Connecting to the bed

The RemoBIF BM must be close (max. 1.5 m distance) to the bed control unit during the entire pairing process.

- **RemoBIF BM**

Press **2** [OK] and **4** [↓] at the same time for 2 sec. so that the display shows „**Pairing - Start with OK**“.

- **Bed**

Unplug the bed's power supply for 10 sec.

- **Connecting RemoBIF BM with the bed**

Plug in the power supply and confirm the pairing process with **2** [OK] within 20 sec.. After about 10 seconds, the RemoBIF BM is connected to the bed.

- **Unlock RemoBIF BM with manual switch**

Go back to the operating mode with **4** [↓].

Place the corresponding magnet on the handset and

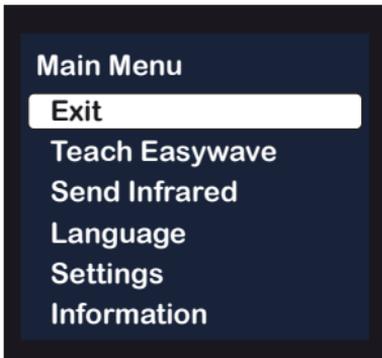
press „Back rest up“  and „Horizontal up“ .

- **Connecting handset with the bed**

If the handset is not (or no longer) connected to the bed, unplug the bed's power supply unit for 10 sec.. Plug in the power supply and confirm „Horizontal up“  and „Horizontal down“  for at least 5 sec. within 20 sec..

## 9. Settings

Press the **2** [OK] button to adjust settings. The display is switched on and the main selection appears:


The RemoBIF BM can be controlled both with Easywave and via infrared.

---

## Teach Easywave

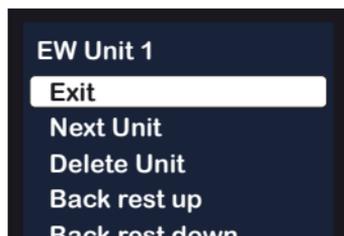EW means Easywave unit. An EW unit is a transmitter unit such as e.g. B. a communication or environmental control device that has an Easywave transmitter. It can also be a simple Easywave handheld remote control or some other special device with an Easywave transmission function. A **maximum of 3 different EW units** can be taught.

„EW unit 1“ shows which EW unit is learned. The complete EW unit is deleted under Delete unit.

The functions are learned one after the other. To do this, the **8 corresponding functions** on the EW unit are actuated and appear on the display: Back rest up, Back rest down, Horizontal up, Horizontal down, Leg rest up, Leg rest down, Foot deep up, Foot deep down.

After each function you have to wait until the Easywave transmitter has been confirmed by a **beep**. After that, the RemoBIF BM jumps to the next menu item to continue.

If an Easywave signal that already exists is learned, this is confirmed with a deep signal tone and the learning process is ended.

 If there is an error, all functions must be learned in again.

## Send Infrared

The infrared codes can be read into the environment control or communication device used via an optional remote control (CSS Art #20324) or directly via the RemoBIF BM. For this purpose, the RemoBIF can send the corresponding IR signals (the IR diode is next to the  display).

The infrared code for programming is sent for the corresponding **8 functions** as long as the  **[OK]** button is pressed: Back rest up, Back rest down, Horizontal up, Horizontal down, Leg rest up, Leg rest down, Foot deep up, Foot deep down.

## Language

The menu language can be set here.

## Settings

- **Drive Cut off**

The running time of the corresponding drive is limited in order not to inadvertently run too far or to the stop if it is operated continuously. Here you can set the maximum time for a continuous function until the forced pause switch is activated.

 Security feature! The factory setting for this is 3 seconds. A change may only be made by a technician who can estimate how long a bed motor can run automatically without causing injury to the user.

---

- ***Pairing***

Look at 8. Installation.

- ***Factory Reset***

At factory reset you can reset all settings.

**Attention:** with this function all settings are lost!

## **10. Maintenance / Review**

The device must be checked for safety and function at regular intervals of 3 months. The following checks must be carried out:

- function
  - plug connector for damage and contamination
  - damage to the casing and other components
-

## 11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to [support@csslabs.de](mailto:support@csslabs.de). Please be sure to include a description of the problem and the serial number.

## 12. Technical specifications

Dimensions: 101 x 101 x 32 mm

Weight: 155 g

IP Rating: IP 54

For bed: Care bed "Dali" from Burmeier

Interfaces: Infrarot, Easywave, Bluetooth® BLE  
(for connection to the bed only)

Power supply: 5V (USB-C), typical 35mA (max. 220 mA while teaching)

Infrared: RC5 Code/38 kHz

### Temperature ranges

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

---

## 13. Conformity / Standards

 This product is CE compliant.

Functional safety according to: DIN EN 60601-1

The product contains a LINAK ABL module.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc. and any use of such marks and logos by LINAK® is under license.

## 14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

---

**THE PASSIONATE ASSISTIVE TECHNOLOGY MANUFACTURER**

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© 2023 CSS MicroSystems GmbH